

Quick-start Guide for Clinicians



Follow these key steps to prescribe reSET and reSET-O, and maximize use of the Clinician Dashboard. Taking note of them now will help ensure a good start for you and your patients as they begin treatment.



Write a prescription by filling out the enrollment form

- Download the form at www.reSETConnect.com or ask your Sandoz Digital Therapeutics Specialist



Complete all required information, including patient's email, date of birth, insurance information, and signature



Fax or email the form to reSET Connect[™] by Pear Therapeutics

- 1-877-256-1320
- PearConnect@PearTherapeutics.com



A patient care specialist may call you for any information that is missing from the form



Complete your account setup by following instructions in the welcome email

- Check your inbox for an email from support@peartherapeutics.com within 24 hours after sending the completed form to us*
- Click the link to create your password
- Log in to your account to get started on the dashboard



A patient care specialist will contact your patient with a unique access code to get started with a digital therapeutic



Use the Clinician Dashboard to gain deeper insights into your patients' progress

- Track lesson completion
- Upload drug screen results as needed
- Follow patient-reported substance use, cravings, triggers, and, if applicable, buprenorphine check-ins
- Reinforce positive behaviors and recovery skills



Patients are encouraged to complete at least 4 lessons per week

Questions? We're on call.

reSET
CONNECTTM
BY PEAR THERAPEUTICS

**Your Patient Service Center for
questions and technical support**



**Call 1-833-MY RESET (1-833-697-3738) Monday-Friday,
8 AM - 6 PM ET, or visit www.reSETConnect.com**